
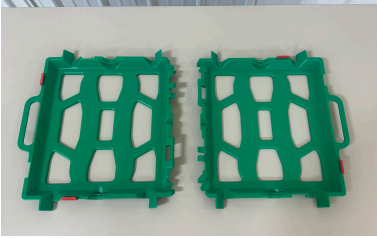
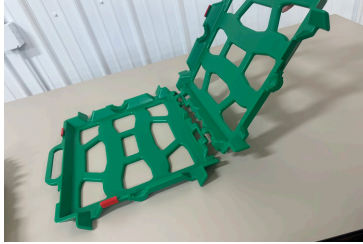








## Use of Colostrum Bags and Cases

		
<ul style="list-style-type: none"> <li>• Our cases nest for easy storage</li> </ul>	<ul style="list-style-type: none"> <li>• To assemble them, place two halves on a flat surface.</li> </ul>	<ul style="list-style-type: none"> <li>• Lift up the one half slightly and press first bottom hinge, then top hinge into place.</li> </ul>
		
<ul style="list-style-type: none"> <li>• Place a new bag flat inside the open case and position the neck in the center of the short side as shown</li> <li>• The two rings of the collar are on the opposite sides of the case</li> </ul>	<ul style="list-style-type: none"> <li>• When the case is closed, the outer ring should be clearly visible below the cap on the outside of the case.</li> <li>• When locked the neck should have no movement</li> </ul>	<ul style="list-style-type: none"> <li>• Make sure all the edges of the bag are inside the case</li> <li>• Lock all four red tabs</li> </ul>

There are three sizes of bags that fit into our cases.

		
<p><b><u>Two liter (2 quart) bags</u></b> To fit two bags inside one case, simply place them opposite of each other, locking their necks in place.</p>	<p><b><u>Three liter (3 quart) bags</u></b> fit without any adjustments.</p>	<p><b><u>Four liter (4 quart) bags</u></b> Fit with the bottom handle folded under. Simply fold the blue section under.</p>